

Feline Blood Typing Kit's report



Serial No.: AZ-190506X

Date: 2019.05.16

Diagnosis kit name: Feline Blood Typing Kit

Date: 2019.09.16-2019.09.24

Report

Specimen:

109 cats had whole blood.

Instruction:

- a) Label each specimen in chronological order.
- b) Use one or more referee blood-typing kit during each test to do a cross check on the blood-type of the specimen to be tested.
- c) Calculate match percentage

Table showing match percentage results of our kit versus other brand

Feline Blood Type	No. of samples	Bioguard		Other Brand		Match (%)
		Type A	Type B	Type A	Type B	
Type A	96	96	-	95	-	98.95
Type B	10	-	10	-	10	100.00
Type AB	3	3	3	3	3	100.00

References:

- 1. Griot-Wenk ME et al. Blood type AB in the feline AB blood group system. Am J Vet Res 1996; 57: 1438-1442.

Referee: _____

Superintendent, Chung-Shan Animal Hospital
Founding member, Taiwan Society of Feline Medicine



Declaration matter

- 1. Do not extract or reproduce this report without the consent of this experiment.
- 2. This analysis report is only responsible for the analysis sample itself.
- 3. The sampler and sample name are provided by the customer.

Address: 242 6th Floor, No. 20-1, Wugong 5th Road, New Taipei Industrial Park, New Taipei City, Taiwan

Tel: + 886-2-55916688 Website: <https://www.bioguard.com.tw/en/>